Hue Light
IMMUNITY CARE SYSTEMS

Whole-body Photobiomodulation Chamber

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04

Precautions

05

Device
Specifications

07

Device
Activation & App

10

Maintenance

11

**Common Issues
Q&A**

12

**Programmable
8-levels
based on the
symptoms
(Dr. Nogier
Frequencies)**

14

**Emergency
use only:
Device
Internal
Control
Function**

**Activation
Method**

15

Warranty

Safety Precautions

[WARNING]

Before using the device, please read and understand the user's manual to ensure correct usage of this chamber. After reading the manual, store it in a safe place for the future reference. The manufacturer assumes no responsibility for any damage caused by mishandling that is beyond normal usage defined in this given manual.

Installation Location

Avoid fires, stoves and damp places. Install in a place with good air circulation. Install at least 20cm away from the wall.

Electricity

This equipment operates on AC 220V (America 110V) power supply and must be supplied with power of 50A (4.5Kw) or higher. See your region and country for the appropriate outlet. Use only grounded outlet.

If you have a medical history or take any medicines, please consult with your doctor or pharmacist to make sure whether your medication is "photosensitive".

[Recommend] Wear a protective glasses during the treatment

The whole body PBM chamber does not have blue light, so it does not have a direct effect on vision, but intense red visible light is emitted. Wearing protective glasses is recommended because the product may dazzle the eye when the equipment is turned on.

[Recommend] Do not exceed daily 20 minutes per use per person

The recommended time for use is from 7~20 minutes per use, depending on the purpose of use, physique and physical condition. Do not use more than once a day, more than 20 minutes per user.

Wash your face and body before use.

Remove all cosmetics, medicines, sunscreens and body lotions that block light transmission and absorption.

Keep the device in cool and less humid place

Keep the temperature around 24~25 degrees and humidity around 40% where the whole body PBM chamber is installed.

Remove all clothing for optimal treatment effect

When using the whole body PBM chamber, it is best to take off the clothes completely.

Product Responsibility

The manufacturer or distributor is not responsible for any problems caused by unauthorized modification or alteration of the equipment.

You must use the equipment only for its intended purpose. If you use it for any purpose other than its intended purpose, you assume all responsibility. The manufacturer/seller is not responsible.

- Abide by the conditions of operation, maintenance and service set by the manufacturer. Maintenance and repair work may only be carried out by a person authorized by the manufacturer or an authorized service company or equipment sales representative.
- For safety and electrical issues, please follow generally accepted safety rules for electrical appliances.
- Safety measures: If there is a problem or the equipment is not in use, disconnect the input power outlet completely.

Device Specification

Product Name	Whole-body Photobiomodulation (PBM) chamber
Model	LLLT
Purpose	Whole-body blood circulation and immunity management use
Power	A/C 220V, 50 ~ 60 Hz, 3.8 kw
Dimension	219.8 cm (L) x 105.3cm (W) x 83.8 cm (H)
Chip standard	LED chip
Quantity	38,880
Wavelengths	520 nm ~ 940 nm
LED module quantity	108 pieces
LED chip temperature	0~35 degrees
PCB material	Aluminum rudder plate
Certification	FDA, CE, & KC

Made in Korea

Manufacturer:
Hue Light Co., Ltd.

Explanation of Product Exterior

Front view

Back view

A/C Connection


Connect the power directly to the ground circuit breaker distribution box as seen in the picture on the right.


Device Activation & App

App Download & Login

This feature only works with Android applications. Please use Android based smartphone or a tablet PC. Connect to Google Play and download "Hue Light PBM"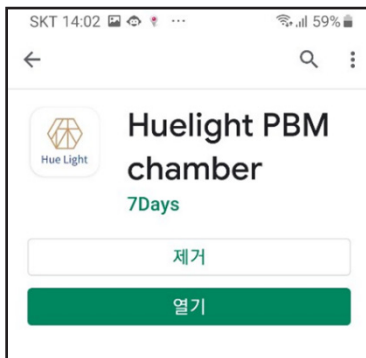

Screen 1.
Google Play Download

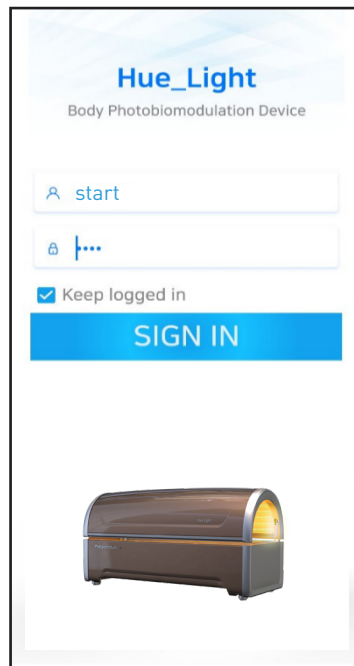Screen 2.
Starting (Login) screen

Enter the provided ID and password for the login.

Device Control

When you log in, you will automatically move to the screen (Screen 3 Device Control) as shown on the right.

The device will be operated only via the app. The app is connected to the device via bluetooth connection, so internet connection is not necessary to operate the device through the app.

Device control Screen Function



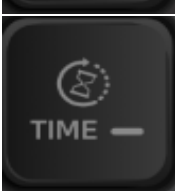
This function is a function to control the machine through the app, you can register the administrator name, ID and password to manage the machine.

If you click (Screen 6)  Admin Management , you can see the registered device manager.

The administrator ID is registered as the purchaser's company name when the machine is shipped. A password is provided separately.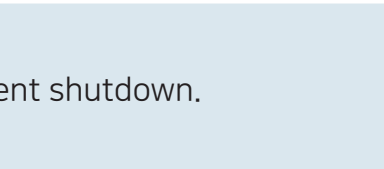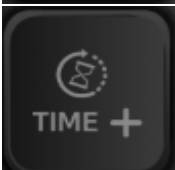


Screen 3

Button	Name	Function	Display screen
	<p>Bluetooth connection</p>	<p>Press and hold this button to display the unique Bluetooth number of the PBM device as shown in Screen 4, and if you click this display number, you will be connected to the Bluetooth and the Bluetooth icon will turn blue.</p>	
	<p>Power on/off</p>	<p>Power on and off, equipment shutdown.</p>	<p>Screen 4</p>
	<p>Time setting +</p>	<p>Increase time setting. 1 minute increase per click up to 20 minutes. It is displayed on the top left as the image on the right.</p>	
	<p>Time setting -</p>	<p>Decrease time setting. 1 minute will be decreased per click. It is displayed at the top left.</p>	
	<p>Nogier Frequencys Setting</p>	<p>This button allows you to select your desired program setting based on Dr. Nogier Frequency A to G. Change the setting by clicking the icon. For Nogier Frequencys A ~ G Function description, see page 12</p>	
	<p>Start PAUSE/Restart</p>	<p>Press this button to start the operation after setting the time and setting the Nogier Frequency. When the machine is in operation, press this button to pause and press again to resume operation.</p>	
		<p>If the machine is started after setting the operating time, the set time will countdown by 1 second.</p>	

App Health Management Function

1. Click the menu bar on the left (as indicated in screen 5) to switch to the right screen and move to each menu.
2. Move to next Screen after selecting  User Management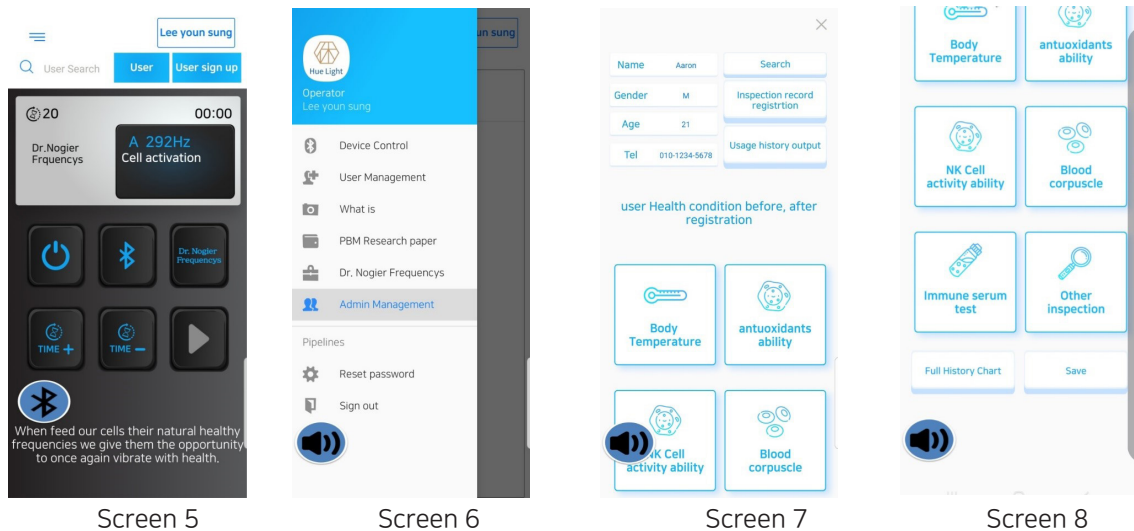

- This function is to register and manage the whole-body PBM chamber users. This should be registered after the user's consent.
- This function is a service that checks the following items at a hospital and registers the result in this app, and checks the graph before and after using the whole-body PBM chamber. (Inspection items are based on valid inspection methods in Korea)
- Therefore, medical law and test method are different depending on country, thus this function is provided if person who orders such function.
- In the future, the chamber will be automatically upgraded to the basic inspection content.
- The following items are the items that generally visualized when using this whole-body PBM chamber. Record these changes before and after, then you can objectively check your blood circulation and health status.

Body temperature change

Measures the body temperature at the tip of a ten finger using a non-contact thermometer. Measurements can be recorded and compared by measuring the body temperature before and after using the whole-body PBM chamber. In general, people with cold hands and feet often experience an average body temperature increase of more than 1 degree Celcius after using the whole-body PBM chamber.

Antioxidant ability

You can register and compare the changed appearance through the blood test at the designated hospital before and after using the whole-body PBM chamber. When used up to 10 times, you can see the increased antioxidant.

NK cell activity

You can register and compare the changed appearance through blood test before and after using whole body PBM chamber. If you have low immunity, such as cancer patients or autoimmune diseases, you can confirm the positive value by comparing the whole body PBM chamber before and after.

Blood cell status

You can register and compare the changes from the whole-body PBM chamber via before and after the blood test. You can see the blood cell condition improved by just one or two times of use.

Immune serum

You can register and compare the changes in your blood before and after using the wholebody PBM chamber. This test mainly checks the state of infection, so it has little to do with before and after using this PBM chamber. However, autoimmune values can be observed with significant changes according to the use of this PBM chamber.

Management and Maintenance

Cleaning after Use

- Clean and disinfect the interior (polycarbonate surface) and exterior of the chamber. Use an alcohol disinfectant cleaner. Do not use detergents that may damage the surface of the interior.
- Sprinkle some alcohol where you want and wipe with a clean, soft cloth.
- Set schedule for daily regular cleaning
- Exhaust fans send air out of the chamber, which can cause dirt to build up over time. If dust accumulates, the exhaust fan should be cleaned. Wipe clean with a soft cloth dampened with water.
- Equipment Inspection: To ensure safe operation, check the device every 300 hours of use.
- Operate the device's functions to check that all devices are properly working.
- Inspect the weight of the opening and closing force of the top cover.
- Inspect all power cords and connections, acrylic and power cords.
- Check the interior. Out of position, broken or badly scratched transparent polycarbonate should be replaced immediately.

Common Issues: Q&A

Symptom	Issues to Confirm	Measures
When the LED light source does not turn on	<ol style="list-style-type: none"> 1. Check power connection 2. LED module contamination status 3. Check for low voltage 4. Check LED lamp life 	<ol style="list-style-type: none"> 1. Check the power of each light source by opening the case at the ends of the chamber 2. Check whether water or contaminants are put into the LED module, and keep it clean with a regular vacuum cleaner. 3. Check the supply voltage 220V 4. Check if the lamp time is over 25,000 hours
LED lamp module does not turn on and timer display is blank	<ol style="list-style-type: none"> 1. Make sure the power cord is unplugged 2. Check each LED connections are connected properly 	<ol style="list-style-type: none"> 1. Make sure the device is connected to a power source. 2. Check the AC power distribution board. Reset circuit breaker or replace fuse
Each LED lamp module is too hot. More than 70 degrees	<ol style="list-style-type: none"> 1. Is there enough airflow? 2. Installation location space temperature 3. Is exhaust function weak because there is much dirt in exhaust fan? 	<ol style="list-style-type: none"> 1. Clean and check the exhaust fan. 2. Always make sure the temperature of the installation location maintained at 25C °.

Programmable Frequencies

7-Dr. Nogier Frequencies

This equipment is applied to optical pulsing with 7-frequency bands which resonate with natural frequencies developed by Dr. Paul Nogier, France. You can program different settings for different symptoms.

Frequency A | 292Hz

Cellular Vitality: Resonates with the Ectoderm (outermost tissue) that forms skin, glands, nerves, eyes, ears, teeth, brain and spinal cord. Assists wound healing, skin and nerve repair, and reduces scar tissue, inflammation and tumors. Try D if a chronic condition related to A is not improving. Frequency A is a universal frequency (as is G) that can be tried to any condition.

Frequency B | 584 Hz

Autonomic nerve balance: Resonates with Endoderm (innermost tissue) that forms the lining of the intestinal tract, the lungs, the bladder, the urethra, and the auditory tube. It also forms the thyroid, thymus, liver, gall bladder and pancreas. Improves nutritional assimilation, allergy problems and balances the parasympathetic nervous system. Used for circulatory problems, edema, and lymph problems.

Frequency C | 1,168Hz

Muscle and blood circulation: Resonates with Mesoderm (middle tissue) that forms connective tissue such as ligaments, tendons, cartilage, muscle, and bone. It also forms the heart, blood and lymph vessels, kidneys, ovaries, testes, spleen, and the cortex of the adrenal gland. Used for muscle, skeletal and myofascial pain.* Try D if chronic condition related to C is not improving.

Frequency D | 2,336 Hz

Mind and body stress adjustment: Helps to balance the two sides of the brain. May also help reduce and treat stress.

Frequency E | 4,672Hz

Peripheral Nervous System: Used for spinal and skin disorders and for pain control. Resonates with the spinal cord and peripheral nervous system. Veterinary experience: Reduces excess calcification such as bone spurs and arthritic joints.

Frequency F | 73 Hz - Harmonic Frequency

Hormone Balance: Resonates with the sub-cortical lower regions of the brain, including the thalamus and hypothalamus—two major control centers for body functions. May also help with muscle spasms, facial pain, headaches and depression. Has been used for non-healing bone fractures and to help balance hormones. Veterinary experience: Improves circulation.

Frequency G | 146Hz - Harmonic Frequency

Brain activation intelligent tissue: Used for memory, psychological disorders, nervousness and worry. Resonates with the cerebral cortex of the brain—involved with thinking, imagining and creating. Veterinary experience: Reduces inflammation and scar tissue on tendons and ligaments, reduces edema and improves gums. Frequency G is a universal frequency (as is A) that can be tried for any condition.

Frquency Cycling Run it sequentially

Seven frequencies from frequency A to G will be turned on sequentially at the set time.


Internal Control Function

For emergency use

There's a control function located on the left side of chamber base. Unscrew and open the cover and you'll see the control display for the device.

Touch screen button

To activate the button functions

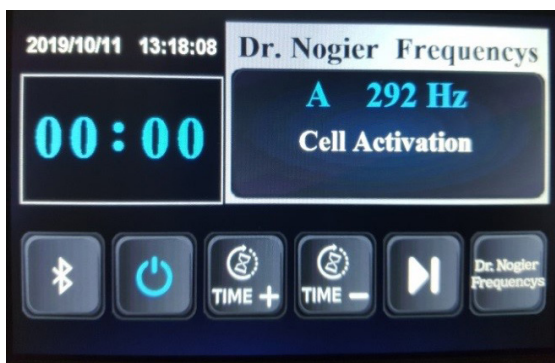screen1

screen2

Button	Name	Function	Display screen
--------	------	----------	----------------



	Bluetooth connection	Press and hold this button to display the unique Bluetooth number of the PBM device as shown in Screen 4, and if you click this display number, you will be connected to the Bluetooth and the Bluetooth icon will turn blue.	
---	----------------------	---	--


	Power on/off	Power on and off, equipment shutdown.	
---	--------------	---------------------------------------	--

	Time setting +	Increase time setting. 1 minute increase per click up to 20 minutes. It is displayed on the top left as the image on the right.	
---	----------------	---	---

	Time setting -	Decrease time setting. 1 minute will be decreased per click. It is displayed at the top left.	
--	----------------	---	--

	Nogier Frequencys Setting	This button allows you to select your desired program setting based on Dr. Nogier Frequency A to G. Change the setting by clicking the icon. For Nogier Frequencys A ~ G Function description, see page 12 Run it Sequentially is a function that runs sequentially from A to G. Normal mode is a mode without pulsing in between the different settings.	
---	---------------------------	--	--

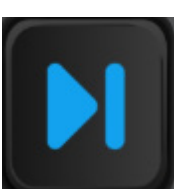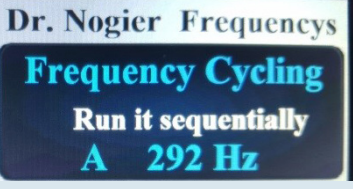	Start PAUSE/Restart	Press this button to start the operation and setting the Nogier Frequency in operation, press this button to pause and resume operation	
---	---------------------	---	---

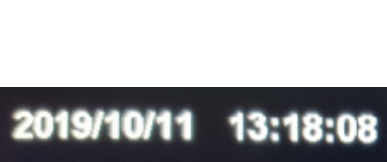	Year Month Day Time Setting: Since the year month date setting is set at the time of factory shipment, when the PCB battery is discharged at about 3~5 years, replace the battery attached to the PCB, and then if you put your finger on this part (Year Month Day Part) of the LCD monitor window, it will switch to the setting button, and set and input.		
---	---	--	--

16

Notes

English

Product Warranty

Product Name	Whole-body PBM chamber		Model Name	LLLT
Manufacturer	Huelight Co., Ltd.		Country of origin	Made in Korea
Free A / S Period			1 year from date of purchase	
Customer	Name		Tel.	
	Address			
Store	Name		Tel.	
	Address			

1. The warranty on this product is covered by the warranty as described in the warranty.
2. If there is any problem with this product, you can get free after-sales service for one year. (In case of paid service guide, the service fee applies even within the warranty period.)
3. Paid Service
 - Product failure or damage due to user error
 - Failure due to improper repair or modification
 - Failure caused by natural disaster
 - Damage caused by failure to observe the precautions specified in this manual.
4. Always show your product warranty when you receive service.
5. Please note that this warranty is not reissued.

1311, E-dong, Gwangmyeong SK Techno Park, 60, Haan-ro,
Gwangmyeong-si, Gyeonggi-do
T. +82-2-898-2116 **F.** +82-2-2083-1353 **E.** info@huelight.kr
 www.huelight.kr

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Hue Light Co. Ltd